# DIABETIC SHOES

## INSTRUCTIONS FOR USE

ORTHOPEDIC/PODIATRIC RETAIL + END CONSUMER/PATIENT

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S.58

S.78

For visually impaired, this document is available in PDF format with this QR code.

## CONTENTS

Introduction .....	6
Product description .....	7
Purpose/Indications .....	8
Potential risks of use and contraindications .....	9
Instructions for use / Assembly instructions for retailers.....	10
Instructions for use for the patient .....	11
Safety instructions and residual risks .....	12
Cleaning and care .....	14
General guidelines .....	16
Technical specifications/Parameters .....	17
Marking .....	18
Disposal/Warranty .....	20

## INTRODUCTION

Dear clients,

Congratulations on the purchase of your new Perpedes diabetic shoe. You have a high quality product selected which one with the utmost care manufactured, tested and packaged.

Please carefully read the instructions for use before using this product. Follow all instructions, particularly the safety instructions.

Perpedes GmbH  
Härtwasen 8-14  
73252 Lenningen  
Germany

T. +49 (0)7021 738 30-0  
info@perpedes.de  
www.perpedes.de

## PRODUCT DESCRIPTION

Perpedes diabetic shoes have been for diabetic patients developed. The models are available in three widths to accommodate feet of different shapes and sizes. The last was developed using English half sizes for a more extensive selection of sizes to ensure the perfect fit for each patient. All models offer sufficient depth to accommodate custommade, diabetic foot orthoses.

All diabetic shoes models have the following features:

- Closed upper
- Sufficient depth for the entire foot
- Low heel height
- Low heel pitch
- Sole with incorporated rocker for forefoot relief and shock absorption
- Padded upper edge
- Rigid / stiff sole
- Padded tongue
- Seamless forefoot area
- Cleanable textile lining
- Velcro fasteners or laces
- Available in 3 widths
- Weather-resistant upper material made of leather
- Non-slip outsole

## PURPOSE/INDICATIONS

Diabetic shoes are designed for diabetic patients. Their main purpose is to reduce pressure points and to achieve even pressure distribution on the sole of the foot.

Perpedes diabetic shoes are appropriate for the following indications:

- Diabetic polyneuropathy (PNP) with pronounced loss of sensitivity
- Pronounced angiopathy (circulatory disorders)
- Existing or healed plantar ulcers
- To avoid ulcer recurrence
- To prevent pressure points and chafing
- Diabetes mellitus with peripheral arterial occlusive disease (PAOD) in combination with loss of sensitivity (PNP)
- Dorsal ulcer caused by ready-made footwear with inadequate toe boxes, e.g. due to pronounced claw / hammer toes

## POTENTIAL RISKS OF USE AND CONTRAINDICATIONS

- Active Charcot neuroarthropathy
- Resection of the first metatarsal
- Extensive toe amputations and transmetatarsal amputations
- Contralateral major amputations
- Motor function limitations / paresis of one or both legs
- Atrophy in hip/knee/ankle joint muscles or joint implant with functional impairment / contracture
- Extreme obesity (BMI > 35)
- Renal failure requiring dialysis
- Highly unsteady gait and standing posture
- Considerable visual impairment that affecting visual perception
- Acute lesions
- Charcot foot
- Bone necrosis

## INSTRUCTIONS FOR USE/ASSEMBLY

### INSTRUCTIONS FOR RETAILERS

Before the first use, the diabetic shoe must be adjusted to the patient's needs by qualified personnel. At the time of delivery, the qualified personnel is responsible for the correct fit and function.

Qualified personnel means any user who is a qualified medical product advisor or any other person authorized under national law to provide and supply this medical product. Product liability cannot be guaranteed if this step is omitted.

The following steps must be carried out by qualified personnel before the first use:

- Determination of the correct shoe length using customary measuring equipment (blue print as well as measuring instruments), as well as the length and width chart provided
- Measurement of the ball girth by means of a measuring tape. Determination of the correct width using the determined values and the sizing chart
- Determination of the area of application in comparison with the clinical picture
- Removal of the shoe fillers and fitting the shoes to the patient's feet
- Adding diabetic foot beds to compensate for swelling or soft cushioned soles. Pay attention to the correct fit
- If necessary, adjustment of the upper material, e.g. of the pressure-distributing, custom made Velcro straps
- If necessary, adjustment of the shoe sole, e.g. compensation of leg length discrepancies, wedges to influence eversion or inversion

- Carrying out a gait and motion analysis
- Providing the patient with instructions for handling and care (laces / Velcro fasteners, daily inspection of the inner shoe for foreign objects)
- Depending on the clinical picture, it is recommended to perform in-shoe pressure measurements with diabetic foot orthoses at regular intervals, if applicable
- Point out that the shoes should only be worn for a few hours at a time at first to get used to them
- The shoes may only be worn with socks

## INSTRUCTIONS FOR USE

Perform the following steps to correctly put on the diabetic footwear; this procedure has to be followed every time. The qualified personnel is required to teach the patient how to handle the product correctly:

1. Open the shoes wide and fold the tongue forward.
2. Check the shoe for foreign objects and remove them before putting on the shoe. Check all parts of the shoe daily for damages.
3. Step into the open shoe.
4. Ensure that the foot is positioned securely / correctly in the shoe, i.e. the heel must be positioned as far back in the shoe as possible. The foot must fit completely into the shoe with a diabetic foot orthosis.
5. Please make sure that the tongue is correctly positioned to prevent pressure and friction.

6. Close the shoe with Velcro straps or laces, depending on the model. The foot needs to be well secured and the shoe shouldn't be too loose or too tight.
7. The therapeutic benefit, the condition of the footwear with regard to possible signs of wear and tear as well as the fit must be regularly checked and assessed by qualified personnel.
8. Get a gait analysis by qualified personnel.
9. Slowly break the shoes in by walking on even surfaces, starting with a few minutes and gradually increase wearing time
10. After taking off the shoe, inspect the feet for possible skin irritations, lesions or other signs of ill-fitting or worn materials. Use a mirror to check the soles of the feet, if needed.
11. The shoes may only be worn with socks.
12. If you notice any unusual changes, please consult your doctor immediately.

## SAFETY INSTRUCTIONS AND RESIDUAL RISKS

Fitting, adjustment and instructions on the diabetic protective footwear must be carried out by qualified personnel only.

The product is intended for use by one patient only.

If you experience pain, swelling, redness, numbness or changes in sensitivity, unusual reactions or discomfort while wearing the shoe, please contact your treating physician or orthotist, pedorthist or similar immediately.

-  If the product is damaged, please replace it immediately or have it repaired by qualified personnel.
-  Depending on the country and its health insurance system, diabetic protective footwear may require a physician's prescription and should be worn under medical supervision. It may only be used in accordance with the instructions in this manual and for the intended purpose.
-  Icy surfaces or a damp outsole can increase the risk of slipping. Be particularly careful when using the product for the first time and get used to it on even and firm ground.
-  After processing the diabetic foot bed, internal pressure measurements must be carried out on a regular basis (follow-up check).
-  Footwear materials are subject to natural aging and the mechanical stress of daily use, which can lead to material fatigue or damage. Therefore, check the shoes and foot orthoses daily for signs of wear.
-  The product must not be used in water and must be protected from moisture (impaired function).
-  Diabetic shoes with a closed upper are weather resistant and withstand splashing water and rain up to a certain level. None of the models is completely waterproof exists.

 The product must be protected from the effects of heat and direct UV radiation. The direct UV radiation (e.g. solar radiation) can lead to a change in color of the diabetic shoe. This change has no detrimental effect on the effect on the function.

 The product must be kept away from fire and heat sources as this may cause material damages.

 Make sure that the product is stored away from moisture.

## CLEANING AND CARE

To prolong the life of the product, dirt should be removed after each use by means of a soft, at most slightly damp brush. Regular impregnation allows protection from moisture and snow to a certain extent.

Do not use any cleaning agents or solvents.

If the inner lining is soiled, it can be wiped off with a damp cloth or mild disinfectant. Please note that the treatment can change the appearance of the shoe. The function and properties will not be impaired. Please test disinfectants on an inconspicuous area first!

Never clean shoes in a washing machine.

Air dry the shoes at room temperature with good ventilation, away from a heat source or direct sunlight.

Sand and salt water can damage the diabetic footwear. Clean the shoes thoroughly if they have been exposed to such conditions.

## **LEATHER**

Leather is a natural product. To prevent the aging of leather, we recommend using good quality leather cleaners and conditioners.

Please note that the treatment can change the appearance of the shoe. The function and properties will not be impaired. Please test care products on an inconspicuous area first. The color of the leather may bleed.

## **SHOE LINING**

It is recommended that the lining and the inside of the shoe are cleaned regularly and disinfected with a mild disinfectant.

## **SHOELACES**

Shoelaces and other fasteners are used daily and are therefore prone to wear out. If necessary, they should be replaced to ensure that the footwear functions and fits properly.

## **VELCRO FASTENERS**

Velcro fasteners must be kept clean to work properly. Clean them regularly and seek replacements for worn fasteners if necessary.

## GENERAL GUIDELINES

Check the diabetic footwear for foreign objects, creases, wear and tear and damage before each use. The functional properties may deteriorate with repeated use (wear and tear). It is therefore impossible to predict the product's lifespan and period of use. The shoes should be replaced or repaired when worn out.

In view of the indication-related selection, the individual adaptations and for hygienic reasons, diabetic shoes may be used by one patient only. It is not possible to reuse the shoes for another patient.

The diabetic shoes can be used in combination with other medical products. However, a combination is only permitted with diabetic foot orthoses that have been selected and adjusted by qualified personnel in accordance with the indication. Modified shoes are regarded as custom-made products.

Please note, leather is a natural product and in addition to color differences may also have defects in the skin structure (e.g. scars), which do not entitle to a complaint about the functioning product. These features are the peculiarities of a natural material and should not be seen as a defect.

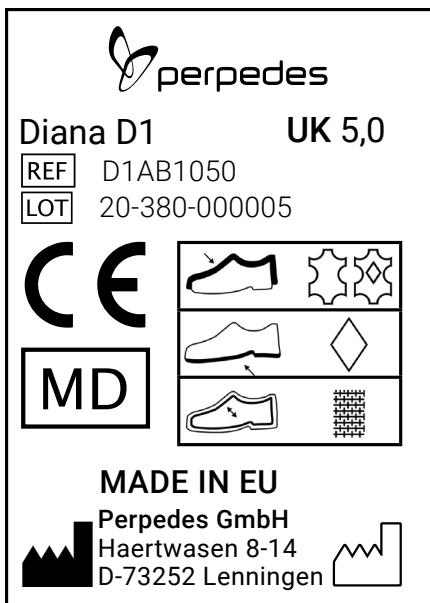
## TECHNICAL SPECIFICATIONS/PARAMETERS

Materials may vary depending on the shoe style and type:

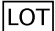
- Upper leather tested for harmful substances
- Upper material made of microfiber or polyacrylic
- Lining made of a combination of polyamide and polyurethane
- Inner soles made of polypropylene nonwoven fabric with an acrylic fiberglass reinforcement
- Outsoles made of ethylene vinyl acetate (EVA), polyurethane (PUR), thermoplastic rubber (TR) or thermoplastic polyurethane (TPU)

## MARKING

Important information and safety-relevant markings can be found on the nameplate on the shoe box. All signs and symbols are explained in the following table.

Illustrations similar

1	Diana	Model designation
2	D1	Width specification
3	<b>UK 5.0</b>	Size indication (UK sizes)
4		Item number
5		Batch designation
6		CE marking
7		Medical device
8		Materials used for upper
9		Materials used for outer sole
10		Materials used for lining & sock
11	 <b>Perpedes GmbH</b> Haertwasen 8-14 D-73252 Lenningen	Manufacturer
12	<b>MADE IN EU</b>	Country of manufacture
13		Date of manufacture

The size and width of each model is displayed on the inside of the tongue.

## DISPOSAL

Product and packaging can be disposed of in accordance with the legal requirements of the corresponding country of use.

## WARRANTY

The currently valid legal regulations apply. Non-compliance with the instructions for use, improperly performed cleaning, unauthorized technical alterations and additions to the product lead to the expiration of the warranty and the product liability in general.

 The CE marking confirms conformity with the relevant requirements of the medical device regulations in accordance with the declaration of conformity. The valid declaration of conformity can be found in the download area on our homepage.

All serious incidents relating to the product shall be reported immediately to the manufacturer and the competent authority.

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